



InnovationScientific

When protection of the healthcare workers matters as much as the health of patients...

Ease of sample collection is pivotal...



InnoScreen™


COVID-19 IgG/IgM Rapid




- Fast result
- Simple operation
- High accuracy
- Screening with confidence

TEST PROCEDURE

For Serum or Plasma Specimens:
Capillary action to collect 5µl serum/plasma accurately

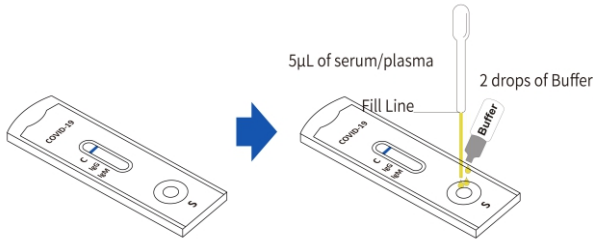


For Whole Blood Specimens:
Easy dropper action to safely pick whole blood from finger prick

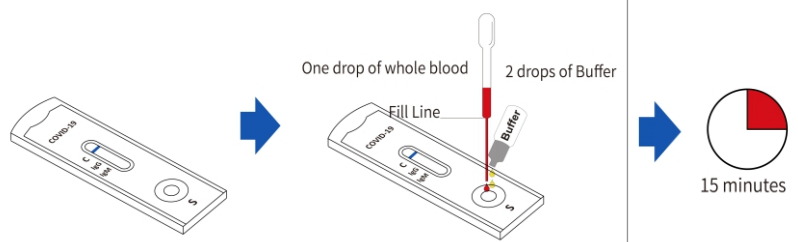

What the PCR and InnoScreen Covid-19 IgG/IgM Rapid Test Results Mean

PCR Test results	Test Results		Clinical Significance
	InnoScreen Covid 19 IgG/IgM Test Result		
	IgM	IgG	
+	-	-	Individual may be in the 'window period' of infection
+	+	-	Individual maybe in early phase of infection, and the body's immune response first produced the antibody IgM, but no IgG was produced or the IgG did not reach the detection limit of the diagnostic reagent
+	-	+	Individual is in late or recurrent stage of infection
+	+	+	Individual is in the active phase of infection, but the human body has developed some immunity
-	+	-	Individual may be in acute phase of infection. The PCR Antigen could be a false negative
-	-	+	Individual may have been infected in the past and is in recovery stage or the virus has cleared the body
-	-	-	Individual is negative for Covid-19


Variation of the Levels of SARS-CoV-2 RNA & Antigen, IgM and IgG after infection.

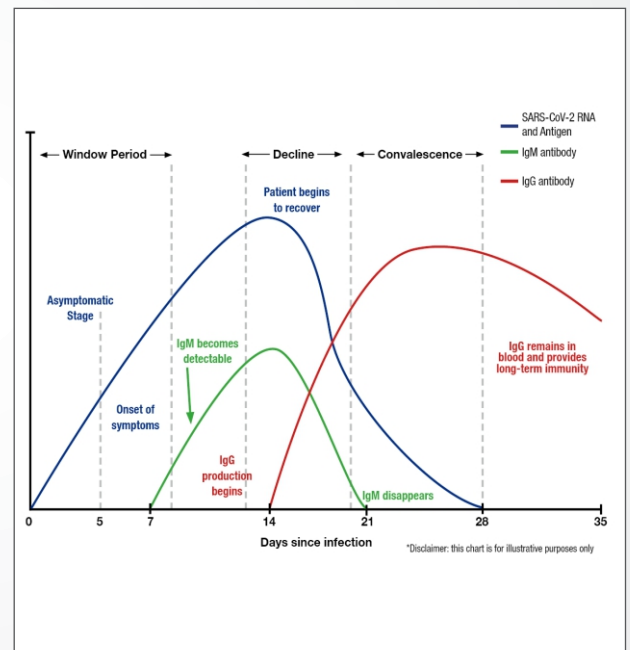

Method		Relative sensitivity	Relative specificity
COVID-19 IgG/IgM Rapid Test	IgM	93.7%	99.1%
	IgG	98.8%	98.7%

Cat. No.	Pack Size	Product Name	Shipping
WCOV-23M	25/Box	InnoScreen Covid -19 IgG/IgM Rapid	Box Size : 15 x 13 x 8 cms. Shipped at Room Temperature

Distributed by:

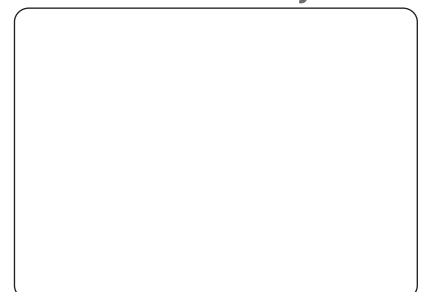

InnovationScientific

Innovation Scientific Pty Ltd.
 5/25 Stoddart Road, Prospect,
 NSW 2148, Australia
 Tel: +61 1300 707 798
 Fax: +61 1300 977 918
 Website : www.innovationsci.com.au
 Email : info@innovationsci.com.au