

COVID-19

Antigen Nasal Test Kit (Nasal Pen)

Non-Intrusive



Benefits

- User-friendly, only 2 pieces to use in process
- More comfortable collection than a traditional nasal swab
- Test specimen totally sealed during process and disposal
- Fast results within 15 minutes- No fuss in no time

Clinical Evaluation

- Relative sensitivity: 98.1% (93.4%~ 99.5%)
- Relative Specificity: 99.8% (98.6%~100.00%)
- Overall Agreement: 99.4% (98.3%~99.8%)

*95% Confidence Interval

Table 1 Clinical Evaluation

	RT-PCR		Total
	Positive	Negative	
Positive	104	1	105
Negative	2	401	403
Total	106	402	508

Test Procedure

